Sample cups ID

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Sample cups ID

Reaction cups for preparation of quantitative in-vitro-determination clinical-chemical and immunturbidimetric tests on SUPER ID clinchem

Order Information

Sample cups ID	0450 980 006 00 / 60675202	100 Reaction cups 1,5ml/500µl + 100 open-end capillaries 10µl
----------------	----------------------------	--

Description

Prefilled reaction cups are prediluted with 500 µl hemolysate-system-solution and are used for hemolysate and diluting the sample at a ratio of 1+50.

Storage

The prefilled sample cups must be stored protected from light and at +5°C to 30°C.

Stability

Unopened reaction cups are stable until the end of the indicated month of expiry if proper storage and handling are observed. Please note the manual of the cartridges!

Specimen

Please note the manual of the cartridges!


Warnings and Precautions

Using reagents of other manufacturer in combination with these prefilled reaction cups is not allowed. The reaction cups will be filled with an accuracy of $\pm 1\%$. Before using the cups check the filling level for significant deviation; visual inspection is sufficient for selection of not correct filled cups. The solution must not differ more than ± 1 mm from level line.

Take the necessary precautions for the use of laboratory reagents.

Preparation and Procedure

The prefilled reaction cups are ready to use. The sample has to be diluted at a ratio of 1 part sample + 50 parts hemolysate-system-solution. The sample preparation is written in the operation manual.

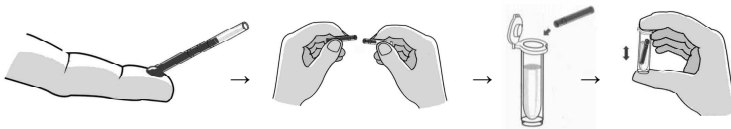Waste management

For disposal please refer to national and local legal requirements.