# Glucocapil reaction cups System Glucose / Lactate / Hb

Reaction cups for preparation of quantitative in vitro determination of glucose, lactate and hemoglobin with the **SUPER GL compact** and **SUPER GL easy +** (enzymatic amperometric/photometric system)

## Order Information

SUPER GL compact, SUPER GL easy +:		
cups	60686200	10 x 100 reaction cups 1.5ml/500µl including Operation Card (60687201)
compatible capillaries	905021001	10 x 100 open-end capillaries 10µl
	901021001	10 x 100 end-to-end capillaries 10µl

## Description

Glucocapil reaction cups are pre-filled with 500µl Hemolysate-System-Solution and are used to *hemolyse* and dilute samples in the ratio 1+50 for glucose and lactate determination on enzymatic amperometric systems *and for the photometric determination of hemoglobin*.

The Hemolysate-System-Solution is filled in yellow cups.

The enclosed Operation Card is required for use with SUPER GL compact and SUPER GL easy +.

## Storage

Glucocapil reaction cups must be stored at 5 – 30°C and protected from light.

## Stability

Unopened Glucocapil reaction cups are stable until the end of the indicated month of expiry. Proper storage and handling of this product *must be observed*. The hemolysed sample can be stored up to twelve hours below 23°C and up to three days at 2 – 8°C.

## Specimen

Capillary blood, venous and arterial blood, serum, plasma and CSF

The blood may contain the following anticoagulants/glycolysis inhibitors: heparin, citrate, fluoride, EDTA. If the sample material is not stabilized, the time between taking the sample and dilution with Hemolysate-System-Solution must not exceed 15 minutes.

## Warnings and Precautions

- Using reagents of other manufacturers *can cause significant measurement errors and, therefore*, is not allowed.
- Please pay attention for the correct combination of Glucocapil calibration solution (green cups) and Glucocapil reaction cups (yellow cups): Other combinations results in wrong measuring results!
- Please refer to the safety data sheets and take the necessary precautions for the use of laboratory reagents.
- For professional use only!

## Preparation

Glucocapil reaction cups are ready to use. Before using the cups check the filling level for significant deviation; visual inspection is sufficient for selection of not correct filled cups. The solution must not differ more than ± 1 mm from printed level line. The sample must be diluted at a ratio of 1 part sample + 50 parts Hemolysate-System-Solution *which is given when using 10µl sample to 500µl in the reaction cup*. Sample preparation is explained in detail in the instruction for use of the device.

Control serums (*except* the pre-diluted GL Control) must be diluted like patient samples.

With the SUPER GL compact / easy +, the Operation Card contained in the package must be inserted into the device according to the instructions for use and remain there.

## Literature

Thomas L, editor. Clinical laboratory diagnostics. 1<sup>st</sup> ed. Frankfurt: TH-Books Verlagsgesellschaft; 1998. p. 136-7.

## Waste management

Please refer to local legal requirements.

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Dr. Müller Gerätebau GmbH
HITADO GmbH