SimplexTAS™ 101 Chemistry Control

SimplexTAS™ Chemistry Control Levels 1 and 2

English

Intended Use

SimplexTAS Chemistry Control is intended for use as a quality control serum to monitor the precision of SimplexTAS 101 systems testing for the analytes listed in this package insert.

Summary and Principle

The use of quality control materials is indicated as an objective assessment of the precision of SimplexTAS 101 system in use and is an integral part of good laboratory practices. Two levels of control are available to allow monitoring of the SimplexTAS 101 system's reliability.

Reagent

This product is prepared from human serum with added chemicals, purified biochemical material (tissue extracts of human and animal origin), therapeutic drugs, stabilizers and preservatives. This product is provided in lyophilized form for increased stability.

Storage and Stability

This product will be stable until the expiration date when stored unopened at 2 to 8°C.

Reconstituted and Refrigerated: When reconstituted and stored tightly capped at 2 to 8°C, this product will be stable for 7 days.

Reconstituted and Frozen: When reconstituted and stored tightly capped at -10 to -20°C, this product will be stable for 30 days.

- Once thawed, do not refreeze this product. Discard the remaining material.
- This product is shipped under ambient conditions (≤ 30°C).

Procedure

This product should be treated the same as patient specimens. Using a volumetric pipet or equivalent, reconstitute each vial with 5.0 mL of distilled or deionized water. Replace the stopper and allow this product to stand for approximately 20 minutes swirling occasionally. Before sampling, gently swirl the vial several times to ensure homogeneity. After each use, promptly replace the stopper and return to the appropriate storage condition. Dispose of any discarded materials in accordance with the requirements of your local waste management authorities. In the event of finding damage to packaging, contact your local distributor.

Limitations

1. This product should not be used past the expiration date.
2. If there is evidence of microbial contamination or excessive turbidity in the reconstituted product, discard the vial.
3. This product is not intended for use as a standard.

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Warning

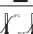
 **Biological source material. Treat as potentially infectious.**

Each human donor unit used to manufacture this product was tested as required by FDA accepted methods. Test results were non-reactive or negative for evidence of infection due to Human Immunodeficiency Virus (HIV), Hepatitis B Virus (HBV), and Hepatitis C Virus (HCV). This product may also contain other human source materials for which there are no approved tests. In accordance with good laboratory practices, all human source material should be considered potentially infectious and handled with the same precautions used with patient specimens.

Analytes

Potassium, Uric acid, Lactate (Lactic acid), Glucose, Creatinine, Urea Nitrogen (BUN), Cholesterol (Total), Cholesterol (HDL), Triglycerides (TG), Creatine Kinase (CK), ALT/GPT, AST/GOT, Gamma Glutamyltransferase (GGT), Amylase, Total Bilirubin (T-BIL)

Symbols

Symbol	Description
	Conformity to the European directive 98/79/EC on in vitro diagnostic medical devices
	In vitro diagnostic medical device
	Reference number
	Lot number
Rev.	Revision letter or number
	Expiry date (year-month)
	Storage temperature. Store at 2 - 8°C (36 - 46°F)
	Manufacturer
	Date of manufacture
	Manufacturer's authorized representative within the European Union
	Warnings and cautions

Control solution information

www.tascom.org
CONTROL SOLUTION INFO

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TASCOM Co., Ltd.
MT Promedt Consulting GmbH