Glucocapil reaction cups System Glucose / Lactate

HITADO

Reaction cups for preparation of quantitative in vitro determination of glucose and lactate with enzymatic amperometric systems of *SUPER G / SUPER GL series*

Order Information

<i>SUPER GL2, SUPER G3, SUPER G(L) speedy:</i>		
cups	60690201	10 x 100 reaction cups 2.0ml/1000µl including Key Card (60691212)
compatible capillaries	905022001	10 x 100 open-end capillaries 20µl
	901022001	10 x 100 end-to-end capillaries 20µl

Description

Glucocapil reaction cups are pre-filled with 1000µl Hemolysate-System-Solution and are used to *hemolyse* and dilute samples in the ratio 1+50 for glucose and lactate determination on enzymatic amperometric systems.

The Hemolysate-System-Solution is filled in colourless cups. The enclosed Key Card is required for use with the SUPER GL 2, SUPER G3 and SUPER G(L) speedy.

Storage

Glucocapil reaction cups must be stored at 5 – 30°C and protected from light.

Stability

Unopened Glucocapil reaction cups are stable until the end of the indicated month of expiry. Proper storage and handling of this product *must be observed*. The hemolysed sample can be stored up to twelve hours below 23°C and up to three days at 2 – 8°C.

Specimen

Capillary blood, venous and arterial blood, serum, plasma and CSF

The blood may contain the following anticoagulants/glycolysis inhibitors: heparin, citrate, fluoride, EDTA. If the sample material is not stabilized, the time between taking the sample and dilution with Hemolysate-System-Solution must not exceed 15 minutes.

Warnings and Precautions

- Using reagents of other manufacturers *can cause significant measurement errors and, therefore*, is not allowed.
- Please pay attention for the correct combination of Glucocapil calibration solution (blue cups) and Glucocapil reaction cups (*colourless* cups): Other combinations results in wrong measuring results!
- Please refer to the safety data sheets and take the necessary precautions for the use of laboratory reagents.*
- For professional use only!*
- In combination with the old SUPER GL/GL ambulance you have to order the required touch memory additionally.*

Preparation

Glucocapil reaction cups are ready to use. Before using the cups check the filling level for significant deviation; visual inspection is sufficient for selection of not correct filled cups. The solution must not differ more than ± 1 mm from printed level line. The sample must be diluted at a ratio of 1 part sample + 50 parts Hemolysate-System-Solution *which is given when using 20µl sample to 1000µl in the reaction cup*. Sample preparation is explained in detail in the instruction for use of the device.

Control serums (*except* the pre-diluted GL Control) must be diluted like patient samples.

For use with the SUPER GL 2 / SUPER G3 / SUPER G(L) speedy, the Key Card contained in the package must be read in only once for each 1000 samples to be measured, in accordance with the instructions for use.

Literature

Thomas L, editor. Clinical laboratory diagnostics. 1st ed. Frankfurt: TH-Books Verlagsgesellschaft; 1998. p. 136-7.

Waste management

Please refer to local legal requirements.

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Dr. Müller Gerätebau GmbH
HITADO GmbH