
HemoCue® Safety Lancet INSTRUCTIONS FOR USE

- Sterile safety blood lancets. For high-quality single-use capillary blood sampling.
- Sterile und sichere Blutentnahme-Lanzette. Ein hochwertiger Einweg-Artikel zur Kapillarblut-Gewinnung.
- Autopiqueurs de sécurité stériles. Pour un prélèvement de sang capillaire de qualité à usage unique.
- Lancetas de seguridad estériles. Para la toma de muestras sanguíneas capilares, de alta calidad y de un solo uso.
- Lancetas estéreis de segurança para retirar amostras de sangue. Para amostragem de alta qualidade do sangue capilar.
- Lancette pungidito sterili monouso indolore. Per prelievo di sangue capillare di qualità.
- Sterila säkerhetslansetter. För kapillär blodprovstagning av hög kvalitet. Engångsbruk.
- Sterile sikkerhedslancetter. Engangslancetter til optimal kapillær blod prøvetagning.
- Steriele veiligheidslancetten van hoge kwaliteit voor capillaire bloedafname. Eenmalig gebruik.
- Αποστειρωμένα νυστέρια ασφαλείας για αιμοληψία. Για υψηλής ποιότητας αιμοληψία από τριχοειδή αγγεία. Για μία μόνο χρήση.
- Steril biztonsági vérvételi lándzsa. Minőségi, egyszeri kapilláris vérvételhez.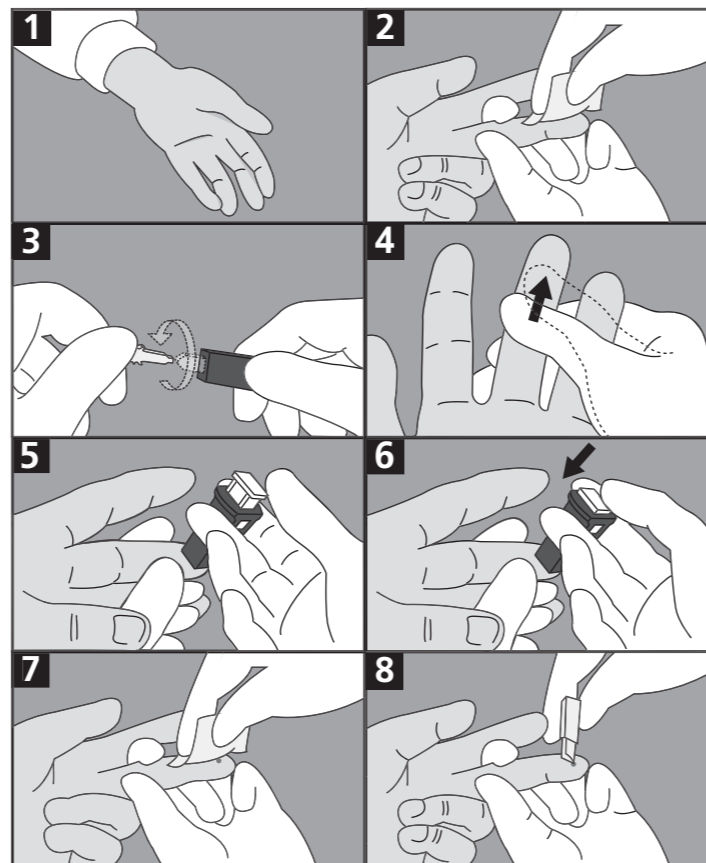

(GB/US) Instructions For Use

1. Make sure that the patient sits comfortably. The hand should be warm and relaxed. It is a good idea to heat cold hands in warm water before sampling. The patient's fingers should be straight but not tense, to avoid stasis.
2. For best results use the middle finger or ring finger for sampling. Avoid fingers with rings. Clean the puncture site with disinfectant and allow it to dry.
3. Twist off the protective tab.
4. Using your thumb, lightly press the finger from the top of the knuckle to the tip. This stimulates the blood flow towards the sampling site.
5. With the thumb's gentle pressure at the tip of the finger, place the lancet at the side of the fingertip. Not only is the blood flow at its best at this point, it also causes the least pain.
6. Press the lancet firmly against the fingertip and activate the white plunger. After use, dispose of the device in a sharps container.
7. Wipe away the first two or three drops of blood. This stimulates the blood flow.
8. If necessary, apply light pressure again until another drop of blood appears. Avoid "milking".

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Asahi Polyslider Co.,Ltd
HemoCue AB