## Glucocapil calibration solution

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Änderungsindex 04  
Stand 2023/03-1  
0500 980 304 00

## Glucocapil calibration solution

Standard for use with Glucocapil-system for quantitative in-vitro-determination of glucose, lactate and / or hemoglobin with enzymatic-amperometric systems

### Order Information

system glucose / lactate	8666 120 01 / 60730401	100 x 1700 µl
system glucose / lactate / hemoglobin	0560 980 002 00 / 0560 980 004 / 60731401	100 x 1700 µl

### Description

The calibration solution is pre-diluted at a ratio of 1 + 50. It can only be used for hemolysate methods with the same pre-dilution.

### Storage

The calibration solution must be stored protected from light and at +5°C to 30°C.

### Stability

Unopened cups with calibration solution are stable until the end of the indicated month of expiry if proper storage and handling are observed. Broached cups must be used within 24 h.

### Warnings and Precautions

Using reagents of other manufacturer in combination with this calibration solution is not allowed. Please pay attention for the correct combination of calibration solution and prefilled sample cups: calibration solution (blue) and prefilled sample cups (nature) for system glucose / lactate and calibration solution (green) and prefilled sample cups (yellow) for system glucose / lactate / haemoglobin. Using the wrong consumption material results in wrong measuring results!

Before using the cups check the filling level for significant deviation - visual inspection is sufficient for selection of the not correct filled cups.

Take the necessary precautions for the use of laboratory reagents.

### Preparation and Procedure

The calibration solution is ready to use, pre-diluted and filled.

For calibration please refer to the user manual for instructions for use.

### Calibration Values / Traceability

The calibration values of calibration solution are traceable and independent from the lot number.

Parameter	Concentration
Glucose	216,2 mg / dl 12,0 mmol / l
Lactate	90,0 mg / dl 10,0 mmol / l

### Waste management

For disposal please refer to national and local legal requirements.

### Literature

Dati F. Reference materials and guidelines for standardization of methods in laboratory medicine. In: Thomas L, editor. Clinical laboratory diagnostics. 1st ed. Frankfurt: TH-Books Verlagsgesellschaft; 1998. p. 1393-1401.

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Dr. Müller Gerätebau GmbH